

NOTICE



Completion time : 30 minutes



MASKIT

- 1 Get a hammer and a pair of scissors. Identify all the items of the kit below.



2x Fabric



1x Metal clip



8x Washer (+2)



1x Press (+1)




1x Elastic




8x Eyelet (+2)

! Do not confuse the press and the eyelets !


- 2 Center the small fabric on the big one. Make sure the holes are aligned. Cut a rectangle in each corner of the bigger fabric. **Do not cut through the holes.**

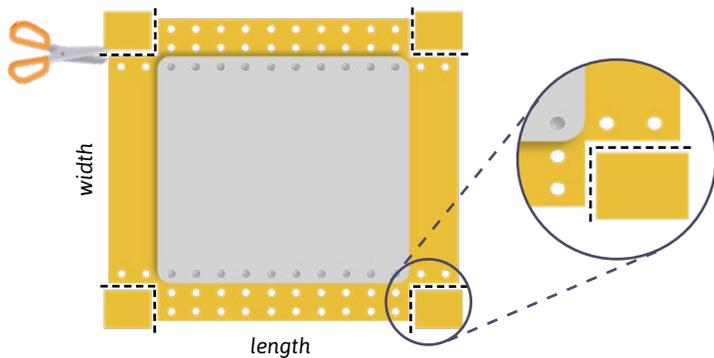

- 3 For each corner, insert an eyelet in the 2nd hole of the small fabric by sliding it through the underside of the big fabric.

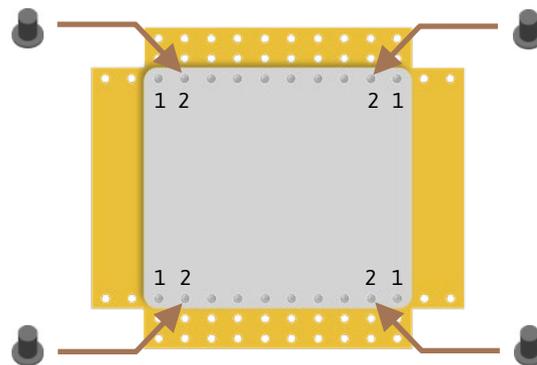


- 4 Fold the fabric **3 times** on itself on its width. Each hole should overlay the next. **Corner by corner**, thread the holes overlapped on the eyelet. If you have difficulties with this, you may thread the holes on the metal clip.

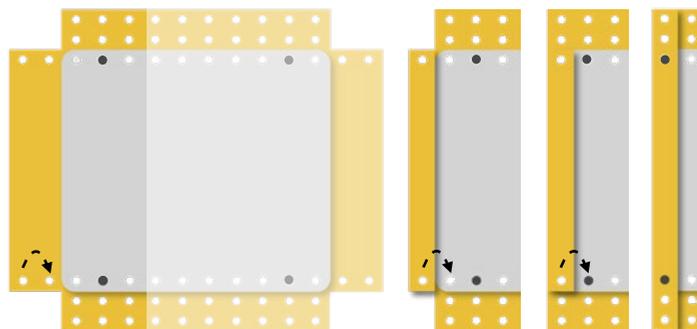

- 5 Repeat step 4 on the other side to reach this :




Check that 6 holes separate the two eyelets.

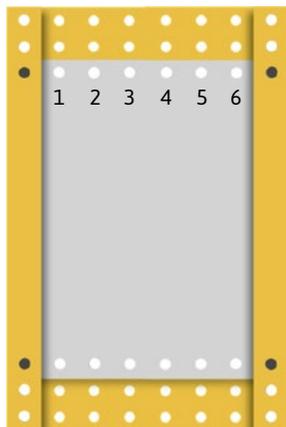


Do not close the eyelets yet.

- 7 Thread the washer (curved side up) over all the layers of fabrics.

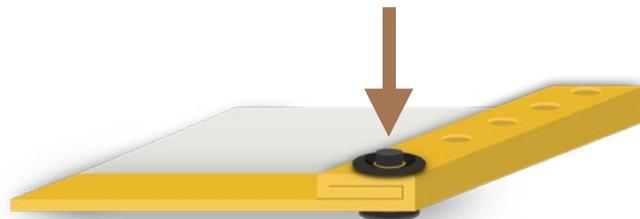


- 8 Insert the press into the eyelet. With a hammer, hit straight on a solid surface. Use also a kitchen board if you have one. Don't hesitate to strike several times. Remove the press when the eyelet is closed.


! Caution, once the eyelet is closed, it's impossible to remove it !

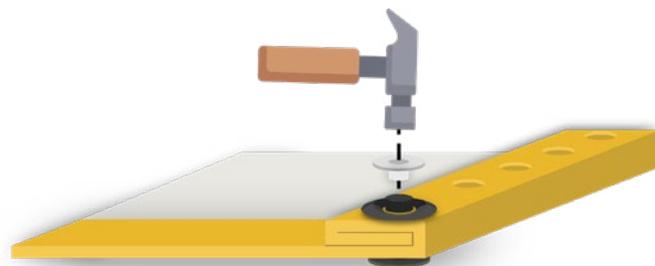


- 6 Pick a corner. **Fold** the fabric **twice** lengthwise. Each hole must overlap to fit over the eyelet.

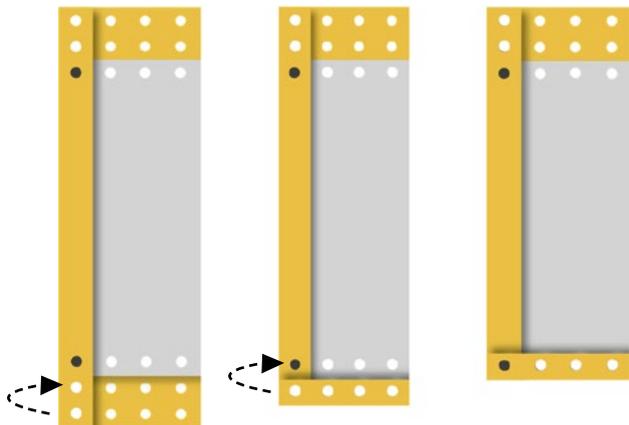


- 9 Close the other 3 corners by repeating steps 6 to 8. Here is the result: 6 holes separating the 2 eyelets on each side.

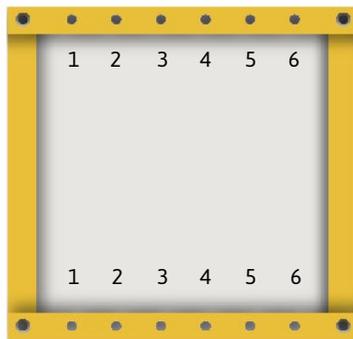


- 10 Complete the central fold. First of all, **check the alignment of all the holes**. Then place an eyelet in the 3rd hole. Fold the fabric so that the 2nd hole is also threaded onto the eyelet.

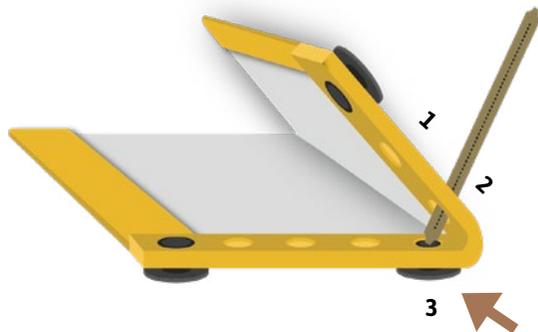

- 11 Fold the fabric in the opposite direction, so that the 1st hole is also threaded onto the eyelet. The 3 holes must be threaded onto the eyelet so that the fabric is folded like an accordion.

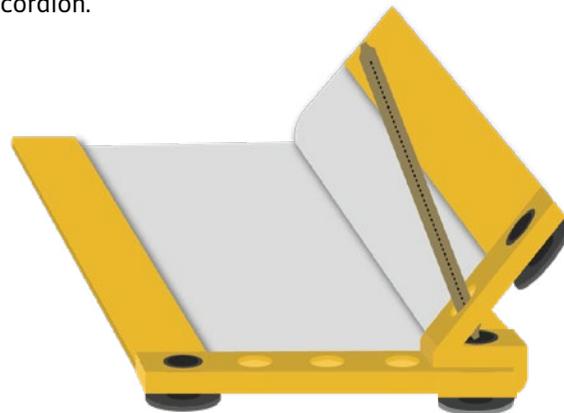


- 12 Check that the fabric is folded as shown below. If it is, close the eyelet just like in steps 6 and 7.

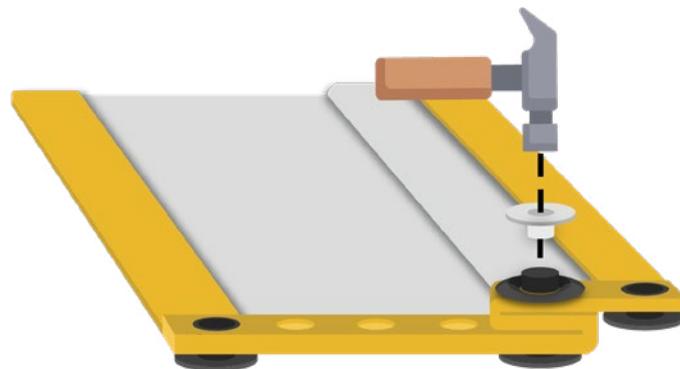

- 13 Make the 3 other folds by repeating steps 10 to 12 in order to arrive as the image below. Strike the 8 eyelets again until they no longer turn into the fabric.


When closed, the eyelet prevents the fabric from fraying.

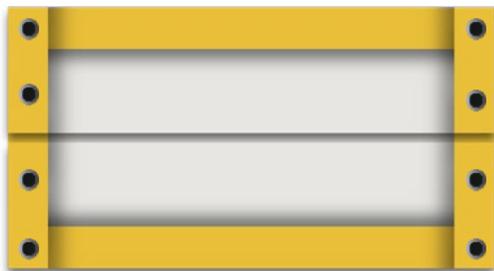


Preparation

MASKIT recommends the following setup:

The elastic band, placed as below, for an easy adjustment and a good fit on the face.

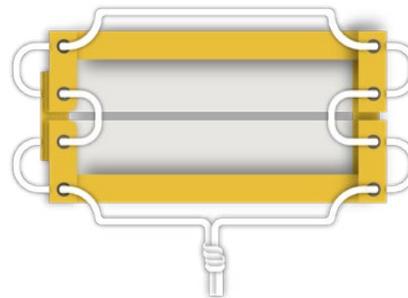

The metal clip as a nose clip. Unroll one of the two folds, place the metal clip and close the fold.

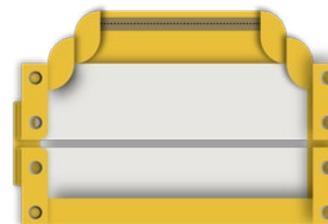

A filter (not included) for ideal protection. The mask remains in the AFNOR's category 2 even without a filter. Open both sides of the mask to place the filter in the center.

INSTRUCTION MANUAL



Put the mask on

1. Wash your hands with soap for 30 seconds.
2. Place the mask over your mouth and nose. Tie the elastic band behind your head. Press the metal clip so that it takes the shape of the nose.
3. Adjust the mask to cover the entire nose. The lower part should be under the chin.



Take the mask off

1. Wash your hands with soap for 30 seconds.
2. Remove the mask by the elastic, do not touch the fabric.
3. Store the mask in a plastic bag or envelope, or wash it directly.



Good practices

Change the mask when wet or dirty.

Avoid touching the outside of the mask. If you do, wash your hands immediately.

The mask can be used several times for up to 4 hours, only if: removed properly and stored in a way that prevents contact.


Do not wear it on your head or under your chin. Always maintain social distancing.



Washing

You can wash the mask in the washing machine at 60°C or boil it for 20 minutes.

Dry the mask immediately after washing.

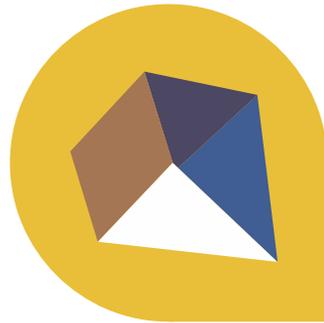

MASKIT



This mask belongs to the AFNOR's category 2 and to NBN/DTD S 65-001:2020.



Check our online video for help building the mask on : www.maskit.be



WWW.MASKIT.BE